THUASNE

SPORT

fr	Bracelet anti-épicondylite Thuasne Sport	3
en	Thuasne Sport tennis-elbow armband	4
de	Thuasne Sport-Armband gegen Epicondylitis	5
nl	Thuasne Sport anti-epicondylitis armband	7
it	Bracciale antiepicondilite Thuasne Sport	8
es	Brazaletes antiepicondilitis Thuasne Sport	9
pt	Braceletes antiepicondilite Thuasne Sport	11
da	Armbind mod epicondylitis Thuasne Sport	12
fi	Thuasne Sport -epikondyliittikyynärvarsituki	13
sv	Thuasne Sport armband mot epikondylit	14
el	Περιογκωνίδα κατά της επικονδυλίτιδας Thuasne Sport	16
cs	Pažní bandáž proti epikondylitidě Thuasne Sport	17
pl	Opaska zapobiegająca entezopatii stawu łokciowego (łokieć tenisisty) Thuasne Sport	18
lv	Ortoze epikondilīta novēršanai Thuasne Sport	20
lt	Apyrankė epikondilitui gydyti „Thuasne Sport“	21
et	Thuasne Sport epikondüliidivastane küünarnukitugi ...22	
sl	Komolčna manšeta proti epikondilitisu Thuasne Sport	24
sk	Epikondylická páska Thuasne Sport	25
hu	Thuasne Sport epicondylitis elleni karkötő	26
bg	Лента срещу епикондилит Thuasne Sport	27
ro	Orteză de cot antiepicondilită Thuasne Sport	29
ru	Браслет при эпикондилите Thuasne Sport	30
hr	Narukvica protiv epikondilitisa Thuasne Sport	32
zh	Thuasne Sport抗上髌炎手环	33
ar	Thuasne Sport سوار المرفق المضاد لالتهاب اللقيمة	35

	1	2
8 cm cm	22 - 30	30 - 38

fr	Circonférence 8 cm sous le coude
en	Circumference 8 cm under the elbow
de	Umfang 8 cm unter dem Ellenbogen
nl	Omtrek 8 cm onder de elleboog
it	Circonferenza 8 cm sotto il gomito
es	Circunferencia de 8 cm bajo el codo
pt	Circunferência 8 cm abaixo do cotovelo
da	Omkreds 8 cm under albuen
fi	Ympärysmitta 8 cm kyynärpään alapuolelta
sv	Omkrets 8 cm under armbågen
el	Περιμετρος 8 εκ. κάτω από τον αγκώνα
cs	Obvod 8 cm pod loktem
pl	Obwód 8 cm poniżej łokcia
lv	Apkārtmērs 8 cm zem elkoņa
lt	Apimtis 8 cm žemiau alkūnės
et	Küünarnuki all ümbermõõt 8 cm
sl	Obseg 8 cm pod komolcem
sk	Obvod 8 cm pod lakťom
hu	Kerület könyök alatt 8 cm-rel
bg	Обиколка 8 cm под лакътя
ro	Circumferință de 8 cm sub cot
ru	Окружность 8 см ниже локтя
zh	肘部下 方 周 长 8 厘 米
ar	محيط بمقدار 8 سم أسفل المرفق

fr BRACELET ANTI-ÉPICONDYLITE THUASNE SPORT

Description/Destination

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THUASNE SPORT TENNIS-ELBOW ARMBAND

Description/purpose

This device is an elbow orthosis providing epicondyle support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

polyamide - polyurethane - polyester - cotton - elasthan - silicone.

Properties/Mode of action

Homogeneous compression on the epicondyle.

Improved proprioception.

Silicone insert.

Removable pad with silicone insert.

Adjustable tightening strap.

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Joint effusions and swelling due to arthritis and arthrosis.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the armband.

Place the adjustment part (tightening strap + buckle) on the inside of the arm.

Adjust the removable silicone insert so that it is correctly positioned over the painful area.

Fasten the strap using the self-fastener.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2034601 (2022-02)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186