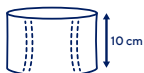
THUASNE

Ortel P


fr	Ceinture pelvienne	5
en	Pelvic belt	5
de	Becken-Stützgürtel	6
nl	Bekkenbrace	7
it	Cintura pelvica	8
es	Faja pélvica	9
pt	Cinta pélvica	9
da	Bækkenbælte	10
fi	Lantiovyö	11
sv	Bäckenbälte	12
el	Ζώνη πυελικής χώρας	13
cs	Pánevní pás	14
pl	Orteza miednicowa	14
lv	Iegurņa josta	15
lt	Dubens diržas	16
et	Vaagnavöö	17
sl	Medenični pas	18
sk	Panvový pás	18
hu	Medence öv	19
bg	Колан за таз	20
ro	Centură pelviană	21
ru	Тазовый бандаж	22
hr	Pojas za zdjelicu	23
zh	骨盆支撐帶	23
ar	حزام للحوض	25

	1	83 - 89	cm
	2	90 - 96	cm
	3	97 - 105	cm
	4	106 - 114	cm
	5	115 - 125	cm
	6	126 - 136	cm


fr	Tour de hanches	Hauteur : 10 cm
en	Hip circumference	Height: 10 cm
de	Hüftumfang	Höhe: 10 cm
nl	Heupomtrek	Hoogte: 10 cm
it	Circonferenza fianchi	Altezza: 10 cm
es	Circunferencia de cadera	Altura: 10 cm
pt	Perímetro de anca	Altura: 10 cm
da	Hoftermå	Højde: 10 cm
fi	Lantionympärys	Korkeus: 10 cm
sv	Höftvidd	Höjd: 10cm
el	Περιφέρεια γοφών	Ύψος: 10 εκ.
cs	Obvod boků	Výška: 10 cm
pl	Obwód w biodrach	Wysokość: 10 cm
lv	Gurnu apkārtmērs	Augstums: 10 cm
lt	Klubų apimtis	Aukštis: 10 cm
et	Puusaümberrõõt	Kõrgus: 10 cm
sl	Obseg bokov	Višina: 10 cm
sk	Obvod panvy	Výška: 10 cm
hu	Csípőbőség	Magasság: 10 cm
bg	Обиколка на ханша	Височина: 10 cm
ro	Circumferința șoldurilor	Înălțime: 10 cm
ru	Окружность бедер	Высота: 10 cm
hr	Obujam bokova	Visina: 10 cm
zh	臀圍	高度 : 10 cm
ar	محيط الوركين	الارتفاع: 10 سم


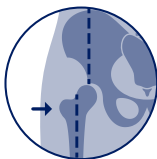
fr	Soutien	Correction biomécanique	Effet proprioceptif
en	Support	Biomechanical correction	Proprioceptive effect
de	Stütze	Biomechanische Korrektur	Propriozeptive Wirkung
nl	Ondersteuning	Biomechanische correctie	Proprioceptieve werking
it	Sostegno	Correzione biomeccanica	Effetto propriocettivo
es	Sujeción	Corrección biomecánica	Efecto propioceptivo
pt	Suporte	Correção biomecânica	Efeito proprioceptivo
da	Støtte	Biomekanisk korrektion	Proprioceptisk effekt
fi	Tuki	Biomekaaninen korjaus	Liikeaistiin kohdistuva vaikutus
sv	Stöd	Biomekanisk korrigering	Proprioceptiv effekt
el	Υποστήριξη	Βιο-μηχανικός έλεγχος	Αποτέλεσμα ιδιοδεκτικότητας
cs	Podpora	Biomechanická korekce	Proprioceptivní účinek
pl	Podtrzymywanie	Korekcja biomechaniczna	Propriocepcja
lv	Atbalsts	Biomehāniskā korekcija	Proprioceptīvs efekts
lt	Palaikymas	Mechaninė biologinė korekcija	Propriocepčinis poveikis
et	Toetus	Biomehaaniline korrektsioon	Proprioceptiivne toime
sl	Podpora	Popravek biomehanske motnje	Proprioceptivni učinek
sk	Podpora	Biomechanická korekcia	Proprioceptívny účinok
hu	Megtámasztás	Biomechanikus korrekció	Proprioceptív hatás
bg	Поддръжка	Биомеханична корекция	Пропрαιοцептивен ефект
ro	Suținere	Corecție biomecanică	Efect proprioceptiv
ru	Поддержка	Биомеханическая коррекция	Пропрαιοцептивный эффект
hr	Potpورا	Biomehanička korekcija	Učinak proprioceptivnog vježbanja
zh	支撐	生物力学矫正	本体感
ar	الدعم	تصحيح ميكانيكي حيوي	مفعول استقبال الحس العميق

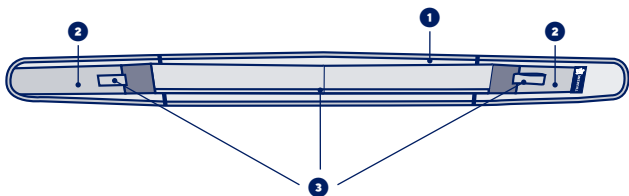(A) (B)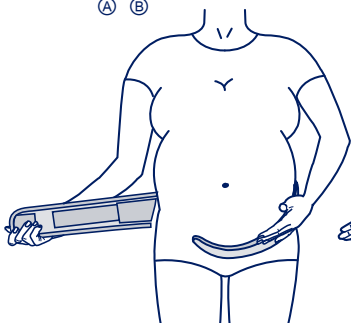

(C)

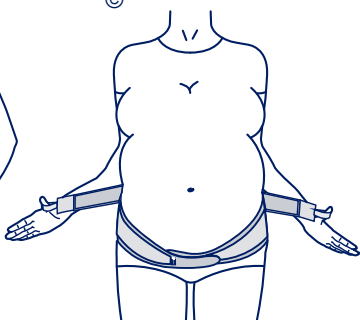(D)

fr

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Description/Destination

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PELVIC BELT

Description/Intended users

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Reinforced elastic fabric. ①

Comfort terry-towelling lining.

Abdominal panels with self-fastening surface. ②

Additional strap with thumb-loops. ③

Textile components: coated elastane - polyamide - polyester.

Non-textile components: polyoxymethylene.

Properties/Mode of action

The device ensures strong support and adaptable compression to the pelvis thanks to the elastic fabric and the additional strap.

Indications

Pregnancy-related pelvic girdle pain (posterior pelvic pain, sacro-iliac pain, symphyseal pain).

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Undesirable side-effects

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the belt around the pelvis. **A**

The belt must be placed as low down on the hips as possible,

The inside label must be positioned upwards.

Fasten the belt, right side over left side, adjusting to the desired compression. **B**

For an adapted support, you can use the additional strap.

Pull the thumb-loops on the additional strap simultaneously and evenly. **C**

Attach the self-fastening parts onto the centre of the abdominal panels of the belt. **D**

The level of support can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing. Machine washable at 40°C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2023302 (2022-06)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros