**THUASNE**

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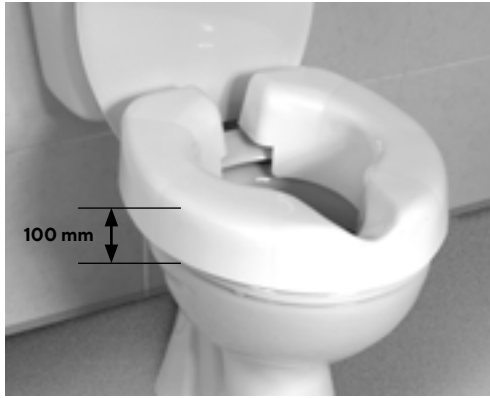
W1510 003 001

<b>fr</b>	<b>SURÉLÉVATEUR CLIPSÉ</b> .....	4
<b>en</b>	<b>CLIP-ON SEAT RAISER</b> .....	4
<b>it</b>	<b>SEDILE ALZA WC A CLIP</b> .....	5
<b>es</b>	<b>ELEVADOR CON GANCHO</b> .....	5
<b>hu</b>	<b>FELCSÍPTETHETŐ WC-ÜLŐKE-MAGASÍTÓ</b> .....	6



1


2

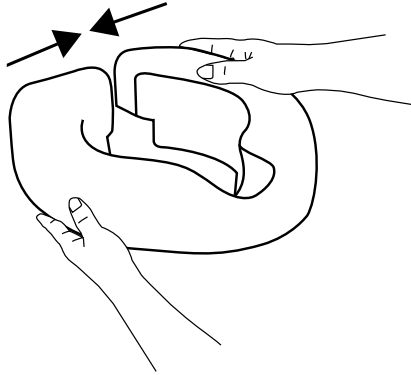3

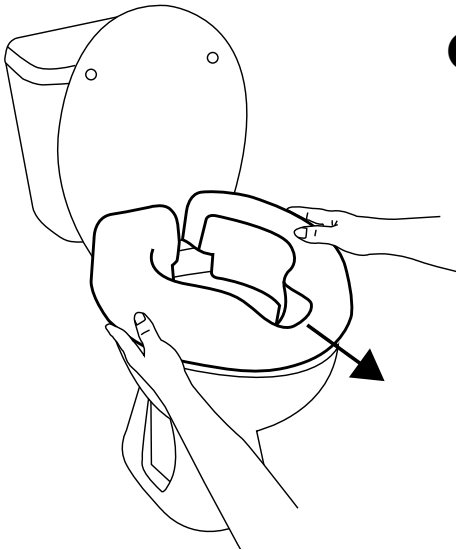4

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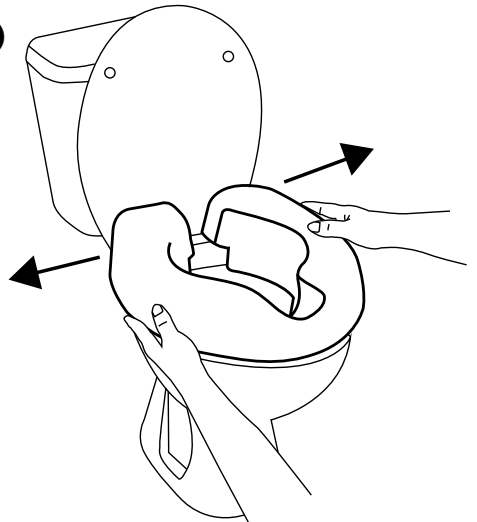## Description/destination

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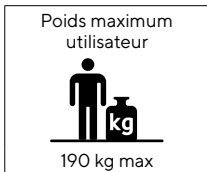
# CLIP-ON SEAT RAISER

## Description/destination

The clip-on seat raiser increases the height of an existing toilet seat by 75 mm, giving a total rise of 100 mm ❶. It provides a safe solution with no need for additional fixings. It can be fitted to most standard toilet seats.

It is designed for people who have difficulty sitting down and standing up from a toilet.

## Precautions

### ⚠ Important warnings

Not suitable for patients weighing > 190 kg. Verify the product's integrity before every use. Do not use the product if it is damaged.

Always make sure the clip-on toilet raiser is clean and that all the clips are properly attached.

To move sideways on the toilet seat raiser, do not grip the raiser as it could become detached.

If you have problems with your balance, seek assistance from someone else to help you sit down and stand up again.

## Instructions for use

Lift the lid of the toilet.

Make sure the existing toilet seat is securely attached.

### Installation

- Press together the two sides of the "horseshoe" opening at the rear. ❷
- Position the front of the clip-on raiser over the toilet seat. ❸
- Position the rear parts over the seat. Once they are in position, release the pressure on the sides. ❹
- Check that the clips are fully engaged by lifting. If the raiser is correctly installed, the existing seat will lift with the clip-on raiser.

### Reusable

The product must be thoroughly cleaned and disinfected before reuse.

## Care

Clean regularly with a mild detergent or a disinfectant solution. This product can be steam-cleaned at up to 80°C and immersed in a hot bleach or antiseptic solution (no hotter than 80°C). Leave to dry before use or storage.

## Storage

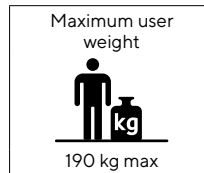
Store at room temperature, in a dry, ventilated place.

## Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033701 (2022-06)